Tiger Dental

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6. Inhalt

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Dear doctor, we are honored that you have chosen this innovative new product from Tiger Dental.

To ensure the safe and effective use of this product, you must carefully read and adhere to this user manual.

The user manual covers all necessary safety instructions but cannot describe all possible circumstances. For this reason, you may find further user notices online at tigerdental.com under the “Know-how” link. Our customer service is available for any additional questions.

The continual development of our products indicates our company’s commitment to innovation. Accordingly, after several clinical uses, we recommend you carefully review the accompanying user manual (this can be found online under the “Downloads” link at tigerdental.com).

1. Manufacturer

Tiger Dental e.U., Belruptstraße 59, 6900 Bregenz, Austria

2. Product description

The unicorn pliers make for easy and pain free removal of cap splint devices. Compared to other removal techniques where the cap is removed from the teeth, the unicorn pliers press out individual teeth from the cap splint through pre-drilled channels, loosening them from the compound. A cap splint can be quickly removed with the unicorn pliers without pain.

3. Use

1. Using a rose-head drill (\varnothing 2.5 mm) to create channels to the buccal tip of each tooth under the cap splint (Fig. 1).
2. Applying the unicorn pliers: with the unicorn at the channel opening and with the gripper jaw at the upper edge of the cap splint (Fig. 2).
3. The individual teeth are pressed out tooth by tooth until the cap splint is fully released (Fig. 3).

4. Care instructions for preparation of forceps and instruments

The use of this product is subject to our “Care instructions for preparation of forceps and instruments (REF 11039) in accordance with the requirements of DIN EN ISO 17664:2004 and the RKI guidelines.

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6. Description of the used symbols of ÖNORM EN ISO 15223-1

1 Creating channels with the rose head drill

2 Pressing out the individual teeth

3 Releasing the cap splint

4 Removed cap splint device

5. References of Quality

Tiger Dental assures the user a flawless quality of its products. The content of this manual is based on experience. The user is responsible for the processing of the products themselves. Tiger Dental is not liable for erroneous results or processing by the user that occur in the absence of influence of Tiger Dental. Although this product is a mature product, improvements and changes can be made, which may not be documented in the present manual. Similarly, we are not liable for printing errors.

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