Tiger Dental

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Unsteril

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Dear Doctor,

Thank you for choosing this innovative new product from TigerDental. To ensure the safe and effective use of this product, please read this user manual carefully and adhere to the instructions.

The user manual covers all necessary safety instructions but cannot describe all possible circumstances. For this reason, you will find further user notices online at tigerdental.com under the "Know-how" link. Our customer service is available for any additional questions.

The continual development of our products indicates our company's commitment to innovation. Accordingly, after several clinical uses, we recommend you carefully review the accompanying user manual (this can be found online under the "Downloads" link at tigerdental.com).

1. Manufacturer

Tiger Dental e.U., Belruptstrasse 59, 6900 Bregenz, Austria

2. Product description

The collar extractor enables safe removal of the collar for adhesive bonding of the Dual-Top™ Jet-screw used with palatal widening devices (MICRO-GNE = Mini-implant collar retained orthodontic expander) using an angle piece. Power of up to 30 cN is produced but with this device it is not transferred to the screw.

3. Use of the collar extractor

- Before use, the thread of the collar extractor should be greased with the enclosed thread grease.
- Then first, carefully remove the plastic above the screw head from the collar using a cylinder cutter (Fig. 2).
- Now place the removal finger of the collar extractor on the underside of the collar (Fig. 3+4).
- Press the mini anchor screw safely from the collar by turning the hexagonal head with the extrusion tool (Fig. 4-6). During removal the collar is securely held in the collar extractor.
- The remaining plastic between the gum plate and the hexagonal plate of the jet screw can be removed with the pliers for collars.

- The remaining jet-screw can be re-used immediately for other purposes (e.g. TopJet Distalizer, Retainer).

4. Contents

REF 10892 Collar Extractor Set (package contents: collar extractor, needle holder, hard metal finishing burr, thread grease)

REF 11124 Hexagonal driver shaft, for collar extractor

5. Care instructions for preparation of forceps and instruments

The use of this product is subject to our "Care instructions for preparation of forceps and instruments (REF 11039) in accordance with the requirements of DIN EN ISO 17664:2004 and the RKI guidelines. All forceps and instruments must be cleaned, disinfected and sterilised if necessary before each use.

6. References of Quality

Tiger Dental e.U. assures the user a flawless quality of its products. The content of this manual is based on experience. The user is responsible for the processing of the products themselves. Tiger Dental e.U. is not liable for erroneous results or processing by the user that occur in the absence of influence of Tiger Dental e.U. Although this product is a mature product, improvements and changes can be made, which may not be documented in the present manual. Similarly, we are not liable for printing errors.

2 Mill out the plastic from the retention collar with the hard metal finishing burr

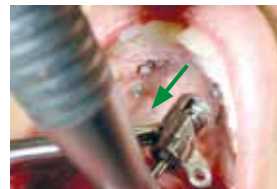
3 Position the collar extractor with the needle holder

4a Correct position, b pressed out collar for adhesive bonding

5 Screwing in the extrusion tool

6 Safe removal of the device with the collar extractor

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