Tiger Dental

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Notizen/notes

A large grid of small dots for taking notes, consisting of 20 columns and 30 rows.

English

Dear doctor, we are honored that you have chosen this innovative new product from Tiger Dental.

To ensure the safe and effective use of this product, you must carefully read and adhere to this user manual.

The user manual covers all necessary safety instructions but cannot describe all possible circumstances. For this reason, you may find further user notices online at tigerdental.com under the "Know-how" link. Our customer service is available for any additional questions.

The continual development of our products indicates our company's commitment to innovation. Accordingly, after several clinical uses, we recommend you carefully review the accompanying user manual (this can be found online under the "Downloads" link at tigerdental.com).

1. Manufacturer

Tiger Dental, Belruptstraße 59, 6900 Bregenz, Austria

2. Product description

The collars for adhesive bondings are stable but easily removable connection elements between the treatment device and the mini anchor screw. The profiled collar is suitable as needed for soldered connections while the straight collar is suitable for laser-welded connections. They are ideally used in combination with the Dual Top Jet screw. These collars for adhesive bonding are used to produce hybrid maxillary expansion appliances and MICRO4/6 expanders.

3. Range of application

Indications: Connection between Dual-Top JS screw (Jet-screw) and treatment device.

Contraindications: Clinically there is no contraindication to the use of a collar in connection with the mini anchor screw. The selection of the associated appliance in each case corresponds to the normal clinical requirements in assessing the required method of treatment. In general, the use of collar-mounted appliances is not advised if the patient's level of mental maturity makes the insertion of mini anchor screws appear to be po-

tentially problematic. The associated procedure for adhering and removing collar appliances may overstress the patient.

Examples of use for collars for adhesive bonding

Hybrid device with mini anchor screws


MICRO2 (purely skeletal, 12-16 years)


MICRO4 (purely skeletal, from 17 years of age)

MICRO-TPA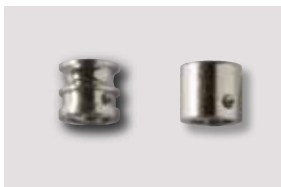

Collars for adhesive bonding profiled/straight

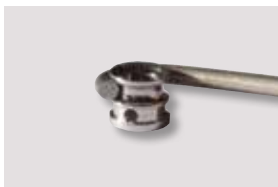
Groove to accommodate wire

Wire butt-welded onto collar

Placement on the Dual-Top Jet screw

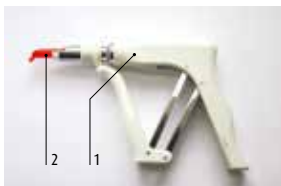Caps dispenser (1) with composite gun (2)

Squeeze composite into the collar

Application of Triad® Gel

Adhered and covered collar

4. Use

In the case of hybrid expanders, the collars are laser-welded on to the anterior supporting arms of the Hyrax screw (straight collar) or soldered on, if necessary (profiled collar). Bands are welded on to the two posterior arms using the classical method. Most of the expansion force and the torque is thus applied directly in the maxilla without placing stress on the canines or premolars. In addition, the hybrid maxillary expansion appliance serves as the perfect placeholder when permanent teeth come in. Another advantage is the fast and secure adhesive bonding of the collar on the mini anchor screw (MAS). After the appliance is inserted, self-curing composite is injected into the precisely sealing collar. This guarantees a load-bearing capacity of up to 400 N. However, later on, the MAS can be easily and painlessly removed in just a few seconds using an efficient extraction device, the collar extractor. As a result, the MAS is not stressed or damaged and can continue to be used.

5. Adhering the collar-mounted expander

For adhesion, the 2-component adhesive Phase II from Reliance is ideally used. For this purpose, two pea-sized amounts are mixed together and then filled into the composite gun (orange) and the rubber plug is inserted.

The composite gun is then placed on the caps dispenser and the device is squeezed until the resin is ready for filling.

Then the composite gun is attached to the collar and, using gentle pressure, a small amount of resin is filled until the excess escapes from the drainage hole (noodle-shaped). Then fill the second collar in the same way. The total processing time for this resin is 120 seconds. The use of a cooled glass plate and a stopwatch is therefore recommended.

Then Triad® Gel is filled into the sleeve from above until a rounded tip forms and no more sharp edges can be felt.

The first activation turns can be made once the resin has been cured. This is normally 2 turns per day (spindle screw 2 turns/day) or PowerScrew (2 sides per day).

Resin and composite removal

Attachment of the collar extractor

Removed collar in the collar extractor

Collar extractor manual (M) or for handpiece (H)

6. Removing the collar-mounted expander

For this purpose, the composite in the upper part of the collar must be removed with a fine hard metal finishing burr (fig. next page) until the uppermost part of the screw head is visible in the collar. In doing so, sufficient water or air cooling should be ensured.

The extraction device can then be placed on the collar such that the finger of the extraction device is underneath the collar, while the rounded spike is screwed into the opening of the collar.

Only a small amount of force is needed to squeeze the screw head out of the collar. Then the second collar must also be detached in the same way. It is possible to slightly loosen one collar after the other; this results in less tension for the patient. Then the bands are removed using band removal forceps.

Then the mini anchor screws can be removed using an appropriate screwdriver.

7. Notices for Single-use Products

This product is nearly ready to use out of the box, but is not sterile and should not be used more than once. The reprocessing of this once used product (recycling) and its reapplication to the patient is strictly prohibited.

8. Safety instructions

This operating and installation manual contains tips and recommendations, but does not release you from personal responsibility! The use of this product is allowed only by orthodontists and dentists according to MDD 93/42/EEC. The user must comply with the provisions of the Medical Devices Directive and any other applicable laws and regulations.

 This product contains nickel.

9. References of Quality

Tiger Dental assures the user a flawless quality of its products. The content of this manual is based on experience. The user is responsible for the processing of the products themselves. Tiger Dental is not liable for erroneous results or processing by the user that occur in the absence

of influence of Tiger Dental. Although this product is a mature product, improvements and changes can be made, which may not be documented in the present manual. Similarly, we are not liable for printing errors.

10. Contents

REF 10841, Collars for adhesive bonding, straight , 4 items

REF 10593, Collars for adhesive bonding, profiled, 4 items

11. Description of the symbols used in ÖNORM EN ISO 15223-1

| | |
|-----------------------------------------------------------------------------------|------------------------------|
|  | Notified body |
|  | Manufacturer |
|  | Date of manufacture |
|  | Lot / Batch number |
|  | Reference / Article number |
|  | Non sterile |
|  | Do not reuse |
|  | Consult instructions for use |
|  | Caution |

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© Tiger Dental, Austria