# Tiger Dental

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**English**

## Dear Doctor,

Thank you for choosing this innovative new product from Tiger Dental. To ensure the safe and effective use of this product, please read this user manual carefully and adhere to the instructions.

The user manual covers all necessary safety instructions but cannot describe all possible circumstances. For this reason, you will find further user notices online at tigerdental.com under the "Know-how" link. Our customer service is available for any additional questions.

The continual development of our products indicates our company's commitment to innovation. Accordingly, after several clinical uses, we recommend you carefully review the accompanying user manual (this can be found online under the "Downloads" link at tigerdental.com).

### 1. Manufacturer

Tiger Dental e.U., Belruptstrasse 59, 6900 Bregenz, Austria

### 2. General product description

Mini Mold, consisting of the Ortho-handle and Mini Mold silicone forms for the individual fabrication of anterior bite ramps.

### 3. Area of application

With Mini Mold, you can fabricate anterior bite ramps for the intrusion of anterior teeth within seconds. The bite ramps are 8.4 mm long and therefore long enough to be easily shortened, if necessary. The included Ortho-handle runs parallel to the bite plane, which guarantees continuously equal results. Small lateral auxiliary lines on the silicone form mark the position of the plateau, which facilitates the alignment of several adjacent bite ramps at the same level. Existing bite ramps can be elongated by simply sliding the Mini Mold over the bite ramp again, if necessary. In order to avoid abrasion, we recommend using unfilled light-curing resins (e.g. Triad®, Heliosit®).

After sterilization, apply a drop of silicone spray to the opening on the Ortho-handle, where the Mini Mold is inserted. The Mini Mold must be inserted into the handle up to the stop! Renew the lateral auxiliary lines on the form with a permanent alcohol marker, if needed.

## Indications

Fabrication of anterior bite ramps for the intrusion of anterior teeth during orthodontic treatment with fixed appliances.

### 4. Instructions for use

#### Shortening bite ramps

With diamond burr and red contra-angle piece

#### Elongating bite ramps

The bite ramp is fabricated as described above. Then, lateral and occlusal retentions are made with a small diamond burr. Fill the Mini Mold only by half and fit it over the existing bite ramp. Cure with light and the job is done!

#### Removal of bite ramps

First remove 90% of the bite ramp with a coarse diamond burr. Remove the rest with a ball shaped carbide finishing burr.

### 5. Warning

The teeth beneath the bite ramps are at risk of caries. Regular checks are necessary! Mini Mold bite ramps should not be left in the mouth for more than 4 months. After this period, the ramps must be removed to check for caries and then be renewed afterwards, if necessary.

### 6. Advice for products intended for multiple use.

The Mini Mold is intended for multiple use. Prior to renewed use, the Mini Mold and the Ortho-handle must be disinfected, cleaned and sterilized. For more details, see „Instructions for the preparation of pliers and instruments“.

### 7. Contents

REF 10609 Mini Mold with Orthoholder, Set

REF 10544 Mini Mold, 2 pcs/pkg

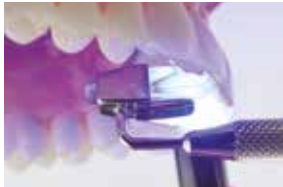
1 Fill

2 Position

7+8 Finally check height of ramps with contact sheet

3 Light cure

4 Pull off and remove excess resin with scaler

9 Anatomically perfect position of bite ramp

10 Several bite ramps form a bite plate

5+6 For further bite ramps please pay attention to the lateral auxiliary line

11 Diamond burr

12 Carbide finishing

Ortho-handle

Silicone form

## 8. Safety Instructions

This operating and installation manual contains tips and recommendations, but does not release you from personal responsibility! The use of this product is allowed only by orthodontists and dentists according to MDD 93/42/EEC. The user must comply with the provisions of the Medical Devices Directive and any other applicable laws and regulations.

## 9. References of Quality

Tiger Dental assures the user a flawless quality of its products. The content of this manual is based on experience. The user is responsible for the processing of the products themselves. Tiger Dental is not liable for erroneous results or processing by the user that occur in the absence of influence of Tiger Dental. Although this product is a mature product, improvements and changes can be made, which may not be documented in the present manual. Similarly, we are not liable for printing errors.

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© Tiger Dental e.U., Austria