Tiger Dental

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Artikelnummer

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English

Dear doctor, we are honored that you have chosen this innovative new product from Tiger Dental e.U.

To ensure the safe and effective use of this product, you must carefully read and adhere to this user manual.

The user manual covers all necessary safety instructions but cannot describe all possible circumstances. For this reason, you may find further user notices online at tigerdental.com under the "Know-how" link. Our customer service is available for any additional questions.

The continual development of our products indicates our company's commitment to innovation. Accordingly, after several clinical uses, we recommend you carefully review the accompanying user manual (this can be found online under the "Downloads" link at tigerdental.com).

1. Manufacturer

Tiger Dental e.U., Belruptstrasse 59, 6900 Bregenz, Austria

2. Product information

The head/chin cap is a removable orthodontic device for the treatment of class III malocclusions (progenie). It is characterised by class III anomalies, typically by a frontal crossbite in the primary dentition (milk teeth) or in early mixed dentition. Frequently, there is also one-sided (unilateral) or bilateral crossbite of the side teeth.


The cause of these dental abnormalities is strong lower jaw growth or a deficit of the upper jaw growth.

3. Areas of application

Head/chin cap therapy is usually begun between the 4th to 7th year of age (Fig. 1a-c). „The earlier, the better“! At this age, the therapy is extremely effective as it makes use of the growth spurt of early childhood (4–6 years of age). “If it grows quickly, it is easy to stop” (Fig. 2). Compliance (cooperation/willingness) is also better at Kindergarten age than later.

4. Patient cooperation

Wearing the chin cap continuously ensures successful treatment! It is possible to wear it for 23 hours, i.e. throughout the day, even in school or

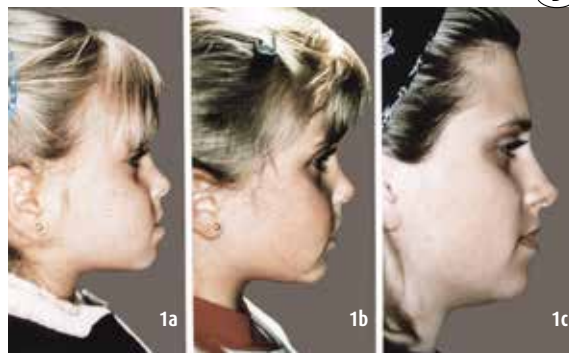1 Profile of patient treated early:
a) Before start of treatment at the age of 4 years
b) 10 months later after chin cap treatment
c) At the age of 17 years

2 Progress of treatment

Kindergarten, apart from sport or hygiene. Small patients usually leave the chin cap off when eating as well. Continuous cooperation is confirmed by indentations in the area of the top of the skull after about 3–4 months of intensive wear (Fig. 3–5). Parents are initially shocked, especially if this has not been explained beforehand. Bony restructuring of the skull means equivalent restructuring of the lower jaw (the gonial angle, the protrusion of the lower jaw over the ascending jaw to the mandibular joint, is “buckled”).

Medical checks can therefore establish without doubt by feeling the cranium whether the patient is really wearing the chin cap. The impressions

disappear by themselves after 2-3 months because of the persistent pressure of brain growth.

Mechanical friction can “rub” the hair to the side. The hair at these sites will grow again afterwards.

5. Construction of the head/chin cap

A plastic bowl is fitted to the chin and linked to a cap on the skull with elastic straps (600 gram module). The side arms of the chin bowl therefore determine the direction of traction. If the pull is too strong, the safety clips spring out of the modules.

6. Adjusting the head/chin cap

1. Try on the head cap and adjust the front band as needed on both sides 0.5 cm to the size of the head with industrial clips. These clips can be removed by straightening them firmly (Fig. 6-8).
2. Hang the safety clip on the head cap using the attached loop until it locks onto the module. Press on the side of the head to bend the safety module (holding clip) (Fig. 9-11) and use the spectacle module if necessary. The head cap can also be turned round with pressure. Module adjustment and loose ends can be fixed in place with elastic bands (the loose end can be cut off in an emergency, if nothing else is possible).
3. Adjust the module to 400 grams per side with the Correx spring balance (recommended treatment) and mark the position. Always mark the force exerted on the elastic band with the safety module, as thick cushioning of the chin bowl creates greater force. The force indicated on the safety module is crucial. The force can be readjusted on one side (holes in the elastic band), to balance the force by cross-tension: readjustment on both sides can lead to too great a force. Always oil the module before handover with a drop of sewing machine oil. This helps stop squeaks!

7. Adjusting the holding/outer arm system

1. The holding arms keep the elastic band away from the skin. Bend the holding arms sufficiently far in from outside. They should just touch the skin with the module attached and should not stick out.

3 Continual cooperation


4 Slight dents in the area of the cranium are a sign of good cooperation.


5 They disappear 4-6 months after the end of chin cap treatment as a result of further brain growth.

6 Before: Pressure on the ear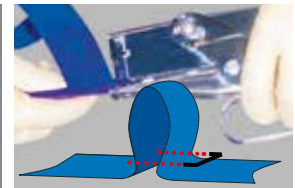


7 Fold in the shoulder strap with the hand stapler and staple the strap together

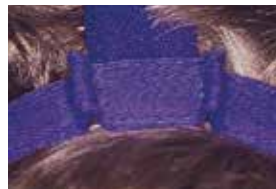8 The stapled strap

9 Afterwards: Ear clear

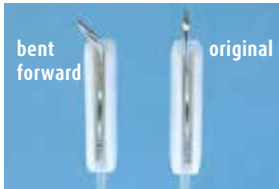10 Bending the safety module

11 Bending bypasses the spectacle

12 Ring-shaped cotton wool pad

13 No pressure on the point of the chin

14 Your pillow says "Thank you".
Left: Competitors, Right: Tiger Dental

2. Gently bend down the outer arms so that the front parts of the chin bowl do not put any pressure at all in the mentolabial area (between chin and lower lip). The chin bowl should form a right angle with the holding arm (90°), i.e. the module lies in the extension to the holding arm. **Never** bend the holding arm **higher**; if it is bent, then bend the holding arm downwards. Otherwise pressure is caused on the teeth and this leads to retrusion (tipping backwards) of the front teeth of the upper jaw and leads to poor pressure sites. Always check the head/chin cap and never hand over the chin bowl the wrong way round.

3. Allow the patients to attach the chin bowl themselves and watch which side the patient prefers. ("Side test") Fix the "weak" side by bending the hook in place and do not change the hook on the "strong" side. The patient should practise attaching and disconnecting. The ends of the holding arms should not scratch and there must not be any risk of injury (the hook shape should be shortened and rounded off).

8. The OPAC (optimized pressure & air-conditioned) chin bowl and fitted inserts

Optimized pressure: Spread the pressure on the surface using a ring-shaped cotton wool pad to protect the point of the chin ("swim ring effect"). The 5x bigger support area means better distribution of pressure and pleasant wear comfort.

Air-conditioned: Very good air circulation is achieved as a result of the openings in the chin bowl, keeping the skin in the chin area dry. The round side of the chin bowl is showing up towards the nose.

Fitted inserts:

- Cotton wool pad cut-outs (size of patient's thumb) as required and should be changed daily.
- Deerskin inserts (particularly kind to the skin)
- Tubular bandage

9. Wearing time

Get used to the chin cap to begin with. Wear the chin cap in the afternoons but **not yet** at night. The joint must also get used to the pressure slowly. After about one week wear the head/chin cap at night as well (increase number of hours gradually). Increase wearing time up to 23 hours daily, according to WITS value (mass for strength of progenie).

10. Care instructions

- Head cap: Machine wash at 30–40 degrees
- Chin bowl: Interdental brush, dishwasher
- Safety module: Sewing machine oil

11. Safety Instructions

This operating and installation manual contains tips and recommendations, but does not release you from personal responsibility! The use of this product is allowed only by orthodontists and dentists according to MDD 93/42/EEC. The user must comply with the provisions of the Medical Devices Directive and any other applicable laws and regulations.

12. References of Quality

Tiger Dental assures the user a flawless quality of its products. The content of this manual is based on experience. The user is responsible for the processing of the products themselves. Tiger Dental is not liable for erroneous results or processing by the user that occur in the absence of influence of Tiger Dental. Although this product is a mature product, improvements and changes can be made, which may not be documented in the present manual. Similarly, we are not liable for printing errors.

13. Contents

REF 10118 Progenie head/chin cap set, black/red

REF 10120 Progenie head/chin cap set, black/lilac

REF 10122 Progenie head/chin cap set, red/light blue

REF 10123 Progenie head/chin cap set, red/black

REF 10124 Progenie head/chin cap set, red/dark blue

REF 10125 Progenie head/chin cap set, light blue

REF 10126 Progenie head/chin cap set, light blue/yellow

REF 10127 Progenie head/chin cap set, dark blue

REF 10128 Progenie head/chin cap set, dark blue/red

REF 10130 Progenie head/chin cap set, lilac/black

REF 10132 Progenie head/chin cap set, pink/lilac

REF 10133 Progenie head/chin cap set, denim

REF 10135 Progenie head/chin cap set, neon yellow/black

REF 10137 Progenie head/chin cap set, neon green/black

REF 10144 Progenie head/chin cap set, light blue/red

REF 10119 Progenie head/chin cap set, black

(Each package contains: 1 x washable head cap, 2 x safety module 600 g, 1 x perforated chin bowl. 10 x ring-shaped cotton wool pads, 1 x deer-skin insert)

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