Tiger Dental

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
Notizen

A large grid of small dots for taking notes, consisting of 20 columns and 25 rows.

English

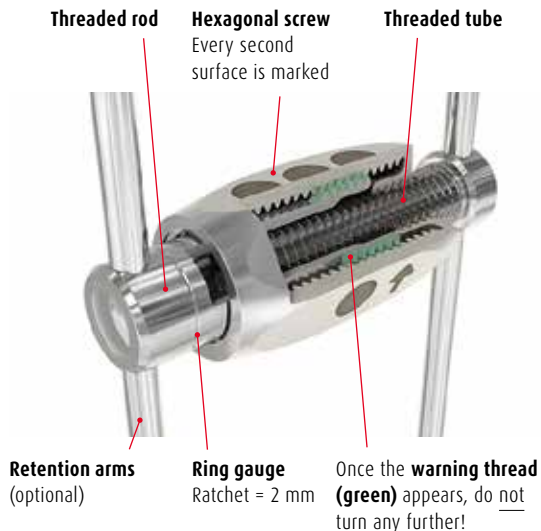The components of the PowerScrew

Technical information

Screw bodies (size)	Max. expansion path	Ring gauge
8 mm	6 mm	3 rings
12 mm	12 mm	5 rings
16 mm	18 mm	7 rings

Expansion per turned surface

Surfaces	expansion path
6	1,0 mm (= 1 turn)
1	0,17 mm

Dear doctor, we are honored that you have chosen this innovative new product from Tiger Dental e.U..

To ensure the safe and effective use of this product, you must carefully read and adhere to this user manual.

The user manual covers all necessary safety instructions but cannot describe all possible circumstances. For this reason, you may find further user notices online at tigerdental.com under the "Know-how" link. Our customer service is available for any additional questions.

The continual development of our products indicates our company's commitment to innovation. Accordingly, after several clinical uses, we recommend you carefully review the accompanying user manual (this can be found online under the "Downloads" link at tigerdental.com).

1. Manufacturer

Tiger Dental e.U., Belruptstraße 59, 6900 Bregenz, Austria

2. Product description

The PowerScrew is a double telescope screw for use in palatal expansion devices. It is available in three sizes (8, 12 and 16 mm installation size).

Activation is simple, as the 8 cm long flat spanner can be quickly and safely used by the patient on the hexagonal screw.

Narrow palates need a small expansion screw. The PowerScrew therefore grows with it. By simply changing the hexagonal screw for the next biggest, the expander can continue to be used without making a new one (see images page).

The PowerScrew and its retention arms are extremely stable and can withstand the greatest forces and torques.

One turn (6 surfaces) equals 1 mm expansion (0.17 mm per surface); every second surface is marked with increasing points.

3. Areas of application

Indications: The PowerScrew is used for skeletal extension of a narrow upper jaw in children, adolescents and adults. It is an expansion screw for dental braces devices (HAAS-type) and dental cap splint devices. It is also used in the Hybrid Expander (partially mini-implant-worn and tooth-worn

By changing the hexagonal screw up to 18 mm expansion possible with one device

PowerScrew 12 and 16 with retention arms

RME devices). In addition, it is used in purely mini anchor screw-worm expanders, like the Micro2 Expander or Micro4 Expander.

4. Use of the Expander and activation of the PowerScrew

As soon as the device is used, medical activation of at least 0.5 mm (3 surfaces) can be carried out. That guarantees initial activation and shows the patient the correct use.

Please give your patient the patient information leaflet with the turning protocol for daily recording. Enter in this turning protocol the turns made by you for activation. We also recommend allowing the patient to make a few turns forwards and back for him/her self. This ensures that the patient has understood the turning process.

Please carry out a withdrawal test at every checkup. Use the Weingart forceps to pull briefly on the retention arms to check the seat and firmness of the apparatus.

5. Expansion, warning thread and turning stop

Each ring gauge corresponds to 2 mm expansion. When maximum expansion of the PowerScrew is reached, the warning thread (green) appears on the threaded tube. The screw should not be turned any further.

6. Securing the PowerScrew after turning stop

After expansion is concluded, light-cured resin (Heliosit® or Triad® Gel) can be applied between the hexagonal screw and the retention arms to fix the hexagonal screw to the thread rod. Further expansion can be carried out after that if required.

7. Notices for Single-use Products

This product is nearly ready to use out of the box, but is not sterile and should not be used more than once. The reprocessing of this once used product (recycling) and its reapplication to the patient is strictly prohibited.

8. Safety instructions

This operating and installation manual contains tips and recommendations, but does not release you from personal responsibility! The use of this product is allowed only by orthodontists and dentists according to MDD 93/42/EEC. The user must comply with the provisions of the Medical Devices Directive and any other applicable laws and regulations.

 This product must be removed before magnetic resonance imaging (MRI). This product contains nickel.

9. References of Quality

Tiger Dental e.U. assures the user a flawless quality of its products. The content of this manual is based on experience. The user is responsible for the processing of the products themselves. Tiger Dental e.U. is not liable for erroneous results or processing by the user that occur in the absence of influence of Tiger Dental e.U.. Although this product is a mature product, improvements and changes can be made, which may not be documented in the present manual. Similarly, we are not liable for printing errors.

10. Contents

- 1× PowerScrew with/without retention arms
- 1× flat spanner

Examples of palate expansion apparatus

Cap splint device

Haas device

Hybrid device with mini anchor screws

MICRO2 (purely skeletal, 12-16 years)

MICRO4 (purely skeletal, from 17 years of age)

Simple activation with flat spanner by the patient himself/herself

Example of treatment

Eight-year-old patient before palate expansion

Cap splint device with PowerScrew 12 11 mm expansion, typical diastema

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© Tiger Dental e.U., Austria