
Tiger Dental

Patienteninformation
Patient information

TopJet ***Distalizer***

TopJet clix Distalizer

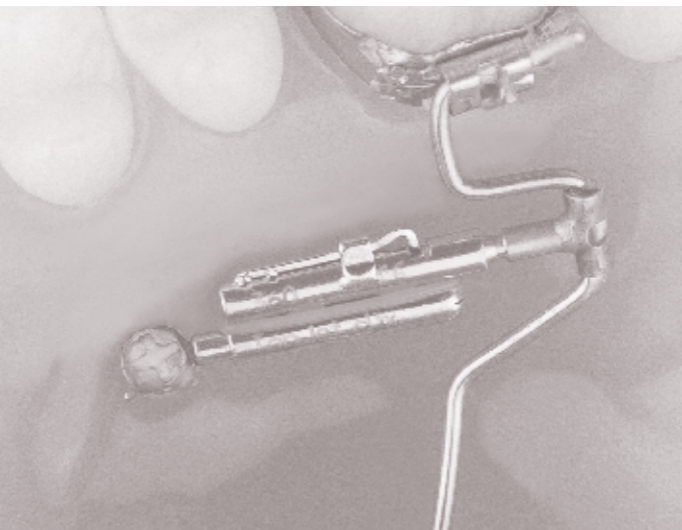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2

Dear patient!

The TopJet Distalizer is an innovative orthodontic appliance for a unilateral or bilateral posterior tooth displacement in the upper jaw. It can be used for adolescent as well as for adult patients. The TopJet Distalizer is placed between the mini-implant and the transpalatal arch. It corrects malocclusions completely and compliance free and therefore you do not need to do anything (Fig. 1).

In the first days after insertion, a feeling of pressure will most likely occur, however, it will decrease with time. It is normal that you will not sense pressure though the treatment is well under way. During the course of treatment you will notice a change in tooth positions in terms of gap opening.

Cleaning the TopJet is very easy and can be done by using a soft toothbrush after every meal. Frequent rinsing of the mouth is recommended (Fig. 2). Please look for gingivitis in this area – if you discover a gingivitis, contact your doctor immediately.

Please chop hard food (apples, carrots and nuts, etc.) before eating. Chewing gum and sticky desserts should be avoided during treatment with the TopJet.

With the TopJet Distalizer you are reaching your treatment goal comfortably, safe and fast. If further questions emerge, please ask your treating doctor.

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